
INSTRUCTIONS FOR USE  
**The McCarthy<sup>®</sup> Mini-Sternotomy Retractor  
& Universal Lift**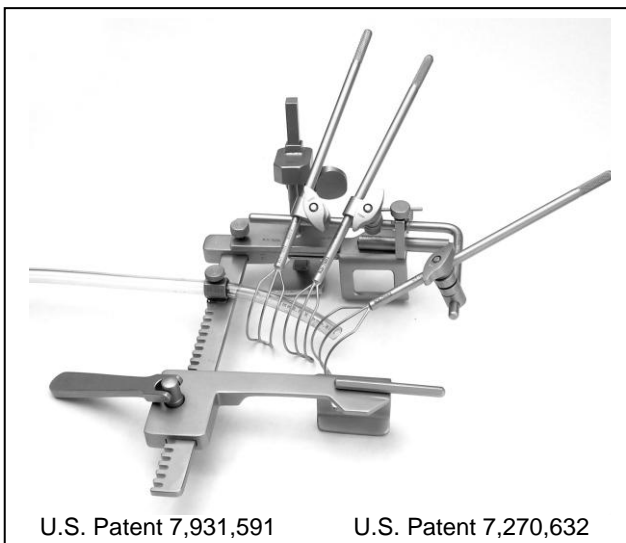

Manufactured by:

**Kapp Surgical Instrument, Inc.**

4919 Warrensville Center Rd.

Cleveland, Ohio 44128, USA

Telephone: (216) 587-4400 or 1-800-282-KAPP

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
Fax: (216) 587-0411

[www.kappsurgical.com](http://www.kappsurgical.com)

e-mail: [info@kappsurgical.com](mailto:info@kappsurgical.com)

**Rx only**

McCarthy is a registered trademark of Kapp Surgical Instrument, Inc.

*Consistent and excellent exposure of the right and left atrium, tricuspid, and mitral valves is fundamental to intracardiac surgical procedures. By using the Kapp McCarthy® Mini-Sternotomy Retractor, the surgeon can gain the optimum exposure necessary for a successful surgical outcome.*

## **DESCRIPTION / INTENDED USE**

This self-retaining retractor is designed to provide excellent exposure for a variety of valve operations, Maze procedure, or other intracardiac procedures. It can also be used for coronary artery bypass operations or for mini-sternotomy procedures depending upon the patient's size. This retractor is designed to allow for better exposure for valve procedures. Recommendations for cleaning and sterilization are included in this manual.

**The device is supplied non-sterile and must be cleaned and sterilized before use.**

## **PRE-SURGICAL ASSEMBLY**

**Figure #1:** Position your chest spreader (A) as shown. Slide the Main Bracket (B) onto either mounting bar. The Main Bracket must have locking screw (C) facing outside of the chest cavity.

**Figure #2:** Insert the long end of the Right Angle Transverse Bar (RATB) (D) into the top hole of the Main Bracket. Secure the bar with the top locking thumb screw tightened against the flat area on the bar.

**Figure #3:** Multiple retractor blades can be attached and positioned as needed along both sides of the RATB, using Universal Clamps (UC). Two styles of UC are provided. The Standard UC must be slid onto the RATB. The Split UC can be added anywhere along the bar after other UC's have been secured in place. Slide the Kapp retractor blade through the hole closest to the wing nut on the UC before mounting it onto the RATB.

**Figure #4:** The **Universal Lift** (E) is attached to the sternal retractor's stationary arm and secured with the locking screw. It is used to lift the patient's sternum 1 to 2 inches (25 to 50 MM) for greater surgical exposure.

**Figure #5:** The fully assembled McCarthy® Mini-Sternotomy Retractor with clip and suction hose in place is now complete.

**Figure #6: Additional feature:** The RATB can be rotated in an upward direction away from the surgical site, by loosening the top locking thumb screw on the Main Bracket. It can then be secured out of the way by retightening the screw on the 2nd flat area on the RATB.

**NOTE: All screws on the Kapp McCarthy® Mini-Sternotomy Retractor are non-removable.**

**NOTE: The retractor blades may be easily repositioned during surgery by loosening the wing nut on the universal clamp(s).**

## **CLEANING**

Immediately after surgery, rinse the instruments under warm, not hot, water to remove all blood, body fluids and tissue. If delays prevent immediate rinsing, soak the instrument in a solution of water and neutral (pH 7) detergent prior to cleaning.

## **ULTRASONIC CLEANING**

Separate the components and place them into the ultrasonic cleaner. Clean per instrument protocol. Rinse the instruments with water immediately after ultrasonic cleaning.

## **AUTOMATIC WASHER STERILIZERS**

Follow manufacturer's recommendations ensuring instruments are lubricated after last rinse cycle and before sterilization cycle.

According to the Robert Koch Recommendations from September 2012, mechanical cleaning and disinfection should be conducted at temperatures in excess of 93° C.

## **AUTOClave STERILIZATION**

Prior to autoclave sterilization, lubricate all instrument parts which have metal-to-metal contact using surgical lubricants such as instrument milk. Do not use industrial lubricants. Instruments must be unlocked before

cleaning to prevent any cracks that may occur from heat expansion during the autoclave cycle. Do not overload the autoclave chamber. The manufacturer's recommended sterilization protocol is:

Sterilizer Type:	Pre-vacuum
Preconditioning Pulses:	3
Minimum Temperature:	132° C
Full Cycle Time:	4 minutes
Minimum Dry Time:	30 minutes
Article Configuration:	Instruments placed in a general surgical case with lid 11 in x 26 in x 3.5 in (28 cm x 66 cm x 9 cm) without brackets or pin mat with the tray wrapped in two layers of 1 ply polypropylene wrap using sequential envelope techniques.

The following cycle should be used when there is an immediate need for the device and there are no other alternatives:

Sterilizer Type:	Pre-vacuum
Preconditioning Pulses:	3
Minimum Temperature:	132° C
Full Cycle Time:	4 minutes
Minimum Dry Time:	N/A
Article Configuration:	Unwrapped
Exposure time:	Minimum 4 minutes

Kapp Surgical Instrument, Inc. confirms that the processing recommendations related to machine cleaning and sterilization by Robert Koch Recommendations from September 2012 may be used. Specifically, Kapp Surgical Instrument, Inc. confirms that sterilization times and temperatures may be extended to 5 minutes and 134° C.

### **MAINTENANCE REQUIREMENTS**

All Kapp McCarthy® Mini-Sternotomy Retractors and attachments should be serviced at least annually. Your retractor should be refurbished by the original manufacturer and original patent holder, Kapp Surgical Instrument, Inc. There is no limit to the number of reuses of this device as long as it is appropriately maintained and serviced by Kapp Surgical Instrument, Inc.

**NOTE:** All screws on the Kapp McCarthy® Mini-Sternotomy Retractor are non-removable. These screws are peened over to prevent the screws from falling into the surgical field.

**DO NOT SEND THIS INSTRUMENT OUT TO BE SERVICED TO YOUR LOCAL SURGICAL INSTRUMENT REPAIR COMPANY. PLEASE SEND ALL REPAIRS AND REFURBISHING BACK TO THE ORIGINAL MANUFACTURER AND PATENT HOLDER:**

**Kapp Surgical Instrument, Inc.**  
4919 Warrensville Center Rd.  
Cleveland, Ohio 44128, USA  
1-800-282-5277 or 216-587-4400

### **INSTRUMENT COMPONENTS**

The Kapp McCarthy® Mini-Sternotomy Retractor is made from instrument-grade quality stainless steel.

**NOTE:** This product is normally intended to be used for less than 60 minutes.

### **WARRANTY**

Kapp Surgical instruments are guaranteed to be free from defects in workmanship and material. Any instrument that proves defective in workmanship or material will either be repaired or replaced at our discretion, without charge.

Kapp Surgical Instrument, Inc. warrants the guarantee of this instrument for the life of this product under the normal surgical use for which it was intended.

Kapp Surgical Instrument, Inc. quality management system is **EN ISO 13485: 2003 Certified.**

**Illustrations:**  
**Illustraties:**  
**Illustrations:**

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Kapp Surgical Instrument, Inc. kvalitetssäkringssystem är **EN ISO 13485: 2003 Certified**.